

USE:

1. Include skin inspections, proper nutrition, turning schedules and daily inflation checks as part of a balanced wound care management program.
2. This product is intended for use as a medical device to aid in the prevention and treatment of pressure ulcers or "bed sores" and general comfort.
3. For optimal clinical benefits to the patient, the mattress should always be properly inflated with the WAFFLE M.A.D. Hand Pump.
4. Do not lift the patient while on the product.
5. For rapid deflation (CPR), pull tab of RED valve.
6. Weight limit: Up to 600 lbs. depending on body type.

MAINTENANCE:

1. Products should be at normal room temperature before inflating.
Store in room temperature of 55 to 85 degrees.
2. Clean all WAFFLE products with soapy water or bleach/water. Do not use cleaning solvents containing alcohol or petroleum distillates. Refer to infectious disease protocol and follow disinfectant manufacturers' instructions. Do not wash or dry in institutional laundry. Proper product inflation and skin care protocol must be maintained to maximize the skin care management provided by these products. Skin inspection, daily hand checks and turning schedules are vital to your skin care program.
3. Use facility procedure for disposing of product and packaging materials.
4. Altitude and temperature may affect inflation.

PLEASE READ:

1. An anti-microbial agent is blended within the material formula of the product.
2. This product has flame resistant qualities in accordance with California Technical Bulletin 129 and ASTM E1590.
3. Do not smoke or use near open flame.
4. This product is not sterile and is not intended to be sterilized.
5. Keep packaging materials away from children.
6. All WAFFLE products are "**Natural Rubber Latex**" **FREE**.
7. All EHOB products are backed by a limited warranty that varies in duration depending on the product. Products found defective in workmanship or materials will be replaced provided failure is not the result of abuse or misuse, alternation or modification, and occurs within the specified warranty period. Tampering with product voids warranty; Example: glue or any unauthorized patching device. Please contact our customer service department for full details. Product warranties are in lieu of warranties for merchantability fitness for a particular purpose and all other warranties expressed or implied.



The Clinical Leader of Medical Devices for Patient Care
 250 North Belmont Avenue, Indianapolis, IN 46222
 Ph. **800.899.5553**
 Fx. **317.972.4625**
 Web: **www.ehob.com**, E-Mail: **corporate@ehob.com**
 ISO 9001: 2008
 U.S. Patent No. 8,281,436 and E.U. Patent No. 2404529



WAFFLE® Mattress Overlay

Medicare Code: EO197, Group I



Indications for use:
Comfort, prevention and healing therapy through Stage IV pressure ulcers. Deep tissue injury protection.

Product	Model	Size	Warranty	WAFFLE M.A.D. Inflation
Expansion Control	1004EC 1004ECP	75" x 30" x 3"	90 Day	35 Complete Strokes
Econo Extended Care Plus	1056EC 1056ECP	75" x 34" x 3"	180 Day	42 Complete Strokes
Econocare Plus	1025EC 1025ECP	76" x 34" x 3"	90 Day	42 Complete Strokes

* Flex-O-Seal design on the 1025/ 1056

Single Patient Use

Pressure, friction and shear are decreased with the WAFFLE Mattress Overlay. The low profile design maximizes the area of contact to equalize pressure and unload the soft tissue at risk.

Skin temperature and moisture are controlled with the pattern of holes*, creating circulating air, dissipating heat and allowing the skin to breathe.




1. UNFOLD



Valve on top and at foot of bed.

2. SECURE

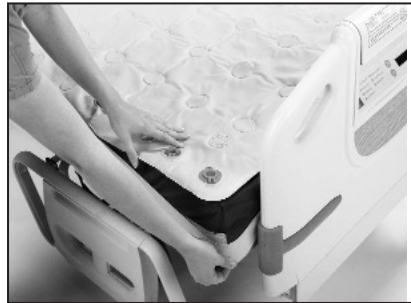

Pull straps under corners.

3. INFLATE (Correct number of strokes located next to clear valve)



Open clear valve and insert pump toward the head of bed and inflate.

Each complete out and in stroke equals one complete stroke.



Has to be forced to fold in half.



Mattress easily lays folded in half.

Four Quick Tips on Your Overlay

1. Overlay should be properly inflated with the WAFFLE MAD Pump.
2. Properly inflated, the overlay will NOT look completely full.
3. Reinflation and fine tuning may require less air.
4. Use RED valve for rapid deflation (CPR).

Hand check for proper air suspension

To check for bottoming out, place flat hand, palm side up, under the overlay beneath patient's buttocks. The ability to do so easily ensures proper inflation.



Make sure that the valves are properly closed after inflation.

Inflation Adjustments

The key to safe patient support is creating the perfect between pliability and stiffness of the mattress. If a mattress is too stiff, the advantages of the surface area are defeated. One inch of air is adequate support for a patient.

If you can slide your hand in and out easily under the overlay, pressure is being relieved. Although you may feel the patient, your hand has displaced the air that has been supporting the patient. The purpose of the hand check is to adjust the air in the product, if necessary. EHOB recommends daily hand checks and then adding or removing air, according to the patient's position and body type.

TURNING



MOBILIZING



TRANSFERRING



Hand Wells

The dogbone-shaped hand wells located on the sides, head and foot of the mattress overlay, provide easier handling to log roll and transfer patients up to 300lbs.

